

Numed Cardiac Diagnostics

Product Support Tech Tip

Tech Tip No: 021-0301
Page 1 of 1

Date first issued: 26 March, 2001
Last Update: 26 March, 2001 10:33

Equipment: Mobil-o-Graph ABP Monitor

Subject: Setting Time and Date without connection to a computer

Information & Recommended Action:

Where no computer is available, the following procedure can be used to set the date and time on the Mobil-o-Graph ABP Monitor.

1. Switch the recorder on and wait for the time to be displayed. The time will be displayed in the format **HH:MM** for hours and minutes.
2. Press and release the **START** and **EVENT** keys *together*. The first digit of the Hours will start to flash. Set the correct first digit of the hours using the **START** key. If no key is pressed within ten seconds of entering the setting mode the unit will return to the normal display mode and this step must be repeated.
3. Move on to the next Hour digit by pressing the **EVENT** key. Repeat step 2 for the second Hour digit. Note there is a short delay before the display moves on to the next digit – don't rush it or you will overshoot to the next digit! If you do overshoot repeatedly press the **EVENT** key until you exit the setting mode and start again from step 2.
4. Press the **EVENT** key again to move to the first Minute digit. Repeat step 2 for setting.
5. Press the **EVENT** key again to move to the last Minute digit. Repeat step 2 for setting.
6. Press the **EVENT** key again. The display will briefly show **DATE** then display **MM:DD** for the Month and Day. Repeat steps 2 to 5 to set the correct month and day of the month.
7. Press the **EVENT** key after setting the last digit of the day and the year will be displayed in the format **:YY**. Set the first digit of the year by repeatedly pressing the **START** key.
8. Press the **EVENT** key and set the last digit of the year.
9. Press the **EVENT** key again and the unit will return to the time display. This completes the setting procedure.

Contact Numed Cardiac Diagnostics Product Support if you require further assistance.