

Numed Cardiac Diagnostics

Product Support Tech Tip

Tech Tip No: 007-0500

Date: 23 May, 2000

Equipment: ABPM for Windows

Noted Symptom: Instruction for Networking

Information & Recommended Action:

To run ABPM for Windows across a network, the following must be performed.

1. Install ABPM for Windows on every PC on the network on which you wish to view the data.
2. On the PC to which the Mobil-o-Graph connects, create a directory called **ABPMDATA**. This directory should be made available to all other users of the network with full read/write access. Move the following files from the **C:\ABPM** directory to the newly created **ABPMDATA** directory:
ABDM.MDB
ABDM.MD0
SPLASH.BMP
3. On every PC that has ABPM for Windows installed, including the one in 2 above, open the file **ABDM.INI**, which is found in the **C:\Windows** directory. Change the line that currently reads **Verzeichnis=C:\ABPM** to read the path name of the new data directory. For example, if the **ABPMDATA** directory is on a PC with a network name **COMPUTER1**, the line will read **Verzeichnis=\\COMPUTER1\ABPMDATA**
Close & save the **ABDM.INI** file.
4. Check the program functions correctly from every PC.

Please contact Product Support if further assistance is required.