


Easy Step User Guide

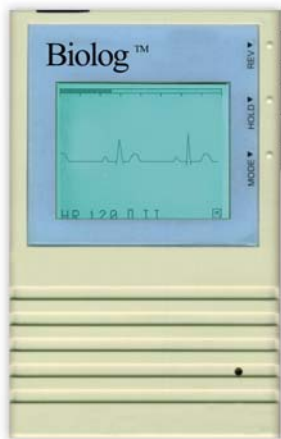

Biolog 3000i

Correct skin preparation is very important.

Prepare the patients skin for electrode application. Apply the sticky electrodes to the skin, connect the crocodile clips to the electrodes.

1. Connect the 12 lead cable to the Biolog ensuring the flat side of the cable is facing toward you. Do not twist the cable as this may damage the pins.
2. Turn the Biolog ON, after a few seconds the Biolog will begin to monitor the ECG, lead II will be displayed on the screen.
3. You can now acquire the ECG by pressing and holding the Mode button until it makes a rising bleep tone.
4. You will now see the ECG acquiring.
5. The procedure is complete when FINISHED is displayed in the screen of the Biolog.

You may now download the ECG onto your Computer.

5. Open the CardioView software on your PC.
6. Click the first icon New Patient.
7. Click the New button to enter new patient details.
8. A progress bar will now appear in the centre of the monitor.

The ECG is now being downloaded. The ECG will be displayed on screen.

9. To view the interpretation press the Details button on the right side of the screen, the details can be edited by a physician in the comments box on the right.
10. To print the ECG, click the printer button on the toolbar,
11. Select the options you require and click OK.